
# EtCO2 Sensor

MEMO


MEMO-Pro

CMZ60MEMO

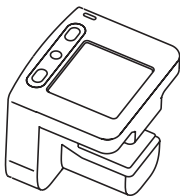
|    |                          |
|----|--------------------------|
| en | Instruction For Use      |
| fr | Mode d'emploi            |
| de | Gebrauchsanweisung       |
| nl | Gebruiksaanwijzing       |
| it | Istruzioni per l'uso     |
| es | Instrucciones de uso     |
| sv | Bruksanvisning           |
| da | Brugsanvisning           |
| no | Bruksanvisning           |
| fi | Käyttöohjeet             |
| pt | Instruções de uso        |
| ru | Инструкция по применению |
| zh | 使用说明                     |
| pl | Instrukcja użytkowania   |
| cs | Návod k použití          |
| sl | Navodila za uporabo      |
| hu | Használati útmutató      |
| tr | Kullanma Talimatı        |


1.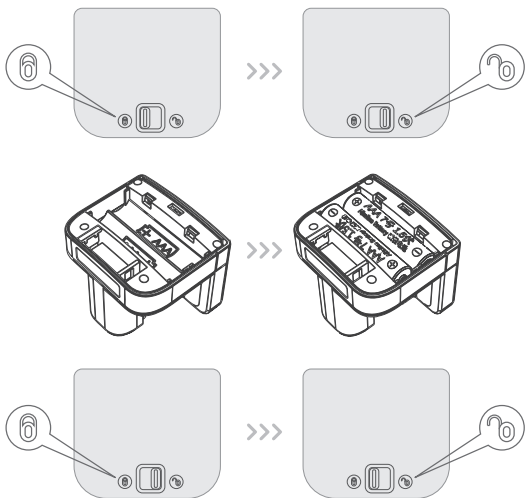


2.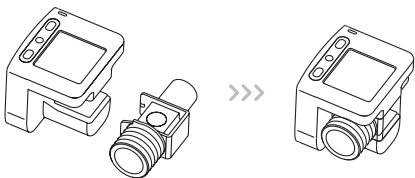


3.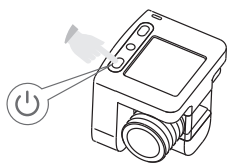

4.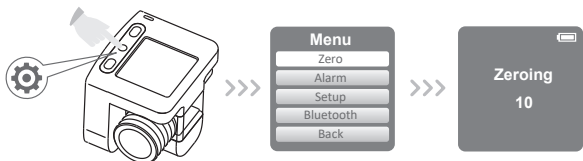

5.

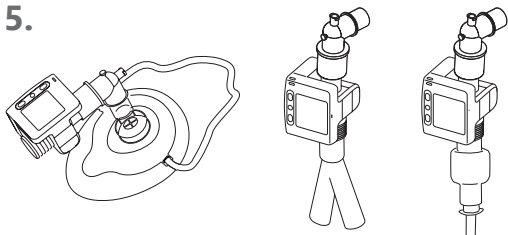## Indications/Contraindications

The EtCO<sub>2</sub> Sensor measures, displays and monitors the partial pressure of carbon dioxide and respiratory rate during anesthesia, recovery and respiratory care. It can be used in the operating room, intensive care unit, patient room, clinic, emergency medicine and emergency transport settings for adult, pediatric and infant patients. It is not intended for use as apnea monitoring; nor during MRI or CT examinations.

## Instructions for use

The EtCO<sub>2</sub> Sensor is for use by trained and authorized medical professionals only. It is used for continuous monitoring of end-tidal carbon dioxide (EtCO<sub>2</sub>) in patients, Respiratory rate (RR) and carbon dioxide respiration waveform (Capnogram).

Open the package to check the product for damage. In case of damage, please contact the manufacturer or local authorized representative.

### 1. Battery installation

Push the battery cover button from  to  and remove the battery cover. Insert two new AAA batteries and reset the battery compartment cover. Then push the button back into place. **(1)**

### 2. Adapter installation

Install the airway adapter to the sensor.

Pay attention to the direction of airway adapter when install it. **(2)**

- Note:**
- Choose adult/pediatric airway adapter or neonatal airway adapter according to patient type.
  - The airway adapter is for disposable and cannot be reused or sterilized, otherwise it will reduce the performance of the system.
  - It is recommended to replace the airway adapter after 2 hours of use. The used adapter should be disposed of in accordance with local regulations regarding biohazardous waste.

### 3. Switch on

Press and hold the power button for 3 seconds to power on. **(3)**

### 4. Zeroing

Press the Settings button to enter the menu interface, select Zero option. **(4)**

Press the Settings button to confirm to start zeroing.

- Note:**
- Zeroing is required before each monitoring and after replacing the adapter.
  - Make sure that the gas in the airway adapter is air with CO<sub>2</sub> concentration close to 0 during zeroing.

### 5. Airway connection

Connect the installed airway adapter into the breathing circuit. **(5)**

### 6. Monitoring results display

-End-tidal carbon dioxide (EtCO<sub>2</sub>)

After the respiration is stable and a complete CO<sub>2</sub> respiration waveform are monitored, the EtCO<sub>2</sub> monitoring results are displayed.

-Respiration Rate (RR)

After three complete respiratory waveforms are monitored, the respiratory rate (RR) monitoring results are displayed.

## Maintainance

### Replace the batteries

The power symbol starts flashing, please replace the batteries. Batteries life depends on the type of battery and environmental factors cannot be accurately predicted.

**CAUTION:** Replaced and used batteries should be handed over to relevant disposal agencies and disposed of in compliance with local laws.

### Cleaning

Wipe the device case with a dry cloth dipped in 75% alcohol or 70% isopropyl alcohol after each use and dry naturally.

**Note:** Do not rinse or sterilize the sensor by immersing it in liquids.

## Specifications

|                                   |   |
|-----------------------------------|---|
| Measuring Principle               | 2 channel NDIR type gas analyzer, No moving parts.  |
| Minimum Sampling Flow             | 1 L/min   |
| CO2 Range                         | 0-99 mmHg, 0-99 kPa, 0-13 %   |
| CO2 Resolution                    | 1 mmHg, 0.1 kPa or 0.1 %  |
| CO2 Accuracy                      | 0~40 mmHg $\pm 2$ mmHg, 41~99 mmHg $\pm 8$ % of readings.<br>0~5.3 % $\pm 0.3$ %, 5.4~9.9 % $\pm 8$ % of readings.<br>0~5.3 kPa $\pm 0.3$ kPa, 5.4~9.9 kPa $\pm 8$ % of readings. |
| Drift of CO2 Measurement Accuracy | Short drift: Less than 1 mmHg offset in 4 hours.<br>Long drift: Meet measurement accuracy requirements within 120 hours.  |
| Respiratory Rate                  | 0~150 bpm   |
| Respiratory Rate Accuracy         | $\pm 1$ bpm   |
| Adult/Pediatric Airway Adapter    | Dead space 6 ml, Flow resistance < 0.3 cm H2O (@ 30 LPM)  |
| Neonatal Airway Adapter           | Dead space 1 ml, Flow resistance < 1.3 cm H2O (@ 10 LPM)  |
| Gas Compensation                  | Automatic atmospheric pressure compensation by default.   |
| Power Supply                      | Two AAA Cell batteries (2*1.5VDC)   |

## Cautions

1. The EtCO2 Sensor is for use by trained and authorized medical professionals only. Please read the instruction manual carefully before use.
2. Before use, please confirm that the airway connection is complete. Do not set up the airway in any way that may cause the breathing circuit and data cable to become entangled or hanged.
3. When the EtCO2 Sensor reaches the end of its life, dispose of electronic waste in accordance with local regulations.
4. Use only with accessories (airway adapter) provided by the manufacturer.
5. Do not use in the following conditions: non-functioning, damaged casing, wet equipment, condensation on the surface.
6. The batteries should be removed when not in use for a period of time.

If you have questions regarding any of this information, please contact manufacturer's technical services department, or local authorized representative.

## Warranty

To obtain information about a warranty, if any, for this product, please contact manufacturer's technical services department, or your local authorized representative.

## Remark

This manual is a simplified manual. For details, please refer to the documentations provided by the manufacturer or complete instructions for use.

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Qinhuangdao Kapunuomaite Medical Equipment S&T Co., Ltd.